



GO SUPRA

Nozzles	Powders	Indications
 AIR-N-GO® EASY SUPRA 120° <i>Supplied as standard</i>	 "CLASSIC" Sodium bicarbonate-based Average particle size 76µm <i>FLAVOURS: Neutral, Cola, Raspberry, Peppermint and Lemon</i>	Finishing after scaling, Preparation for whitening, Cleaning of inter-proximal areas, fissures and crevices, Preparation of the tooth surface before etching, Cleaning of orthodontic brackets, Removal of supra-gingival biofilm, plaque, stains and temporary cement residues
	 "PEARL" Calcium carbonate-based Average particle size 55µm <i>FLAVOUR: Neutral</i>	
 AIR-N-GO® PERIO EASY <i>Supplied with the PERIO kit as an option</i>	 "PERIO" Glycine-based Average particle size 25µm <i>FLAVOUR: Neutral</i>	Removal of the sub-gingival biofilm Treatment of periodontal disease and peri-implantitis Pockets 3mm - 8mm
 AIR-N-GO® PERIO <i>Supplied as an option</i>		Treatment of periodontal disease and peri-implantitis Pockets 8mm - 10mm
 AIR-N-GO® PERIO MAINTENANCE <i>Supplied as an option</i>		Periodontal maintenance Pockets up to 4mm

GO PERIO
Teeth & Implants

For more information about good practice and the use of AIR-N-GO® easy, see our **YouTube** channel



COMMERCIAL OFFER

Standard

AIR-N-GO® easy: AIR-N-GO® easy handpiece with adaptor depending on the model (KaVo*, Sirona*, W&H*, Bien-Air*, Midwest*), 1x SUPRA 120° nozzle, 1x intro kit of 10 multi-flavour sticks of "CLASSIC" powder and 2x sticks of "PEARL" powder, 1x AIR-N-GO® easy maintenance kit
 *KaVo, Bien-Air, W&H, Sirona and Midwest are registered trademarks.

Optional

PERIO easy Kit: AIR-N-GO® PERIO easy nozzle and 1x bottle of 160g AIR-N-GO® "PERIO" glycine powder
Supplied alone: AIR-N-GO® PERIO nozzle
Supplied alone: AIR-N-GO® PERIO Maintenance nozzle

Non contractual document - Ref. D10231 - V3-04/2016 - Copyright © 2015 SATELEC. All rights reserved. No information or part of this document may be reproduced or transmitted in any form without the prior permission of SATELEC.



All clinical applications are easy with the all-in-one air polisher



AIR POLISHING IS BECOMING EVEN SIMPLER WITH



Bacterial decontamination
 Freshness
 Active control
 OF PERIODONTAL DISEASE
 Reliability
 Supra
 Perio
 Freedom of movement
 Simplicity
 Design

DUAL FUNCTIONS

ACTEON® EQUIPMENT has created the new generation of portable air polishers, AIR-N-GO® easy with dual functions:

- "SUPRA" function" for supra-gingival polishing: daily prophylactic polishing.
 - "PERIO" function" for sub-gingival polishing: maintenance, periodontal and peri-implant treatments.
- This medical device allies simplicity with performance.



COMPLETE SOLUTION

4 nozzles for active control of the progress of periodontal disease.

Depending on your clinical needs, choose:

- Sodium bicarbonate-based "CLASSIC" powders
- Calcium carbonate-based "PEARL" powder
- Glycine-based "PERIO" powder

SINGLE HANDPIECE

A single handpiece to cover a range of care and treatment

AIR-N-GO® easy combines essential functions with reliability to facilitate your everyday life:

- Fast start-up: direct connection to the high speed handpiece connector.
- Greater freedom of movement: 360° rotation, fast and accurate treatment of areas requiring care.
- Easier cleaning of the handpiece: main parts can be dismantled.

Treatment of peri-implantitis
 Compatibility
 Patient safety
 Power and performance
 Easy to convert
 Prevention
 Flexibility
 Comfort

YOUR PARTNERS IN PREVENTING AND FIGHTING DISEASE

AIR-N-GO® nozzles, which result from the most advanced fluid dynamics modelling techniques, provide continuous, accurate and controlled spray whatever the clinical context.

In daily practice these nozzles provide power and efficiency for the removal of supra- and sub-gingival biofilm.

AIR-N-GO® SUPRA NOZZLE *(standard)*

- Tool suitable for any supra-gingival prophylactic treatment in combination with AIR-N-GO® "CLASSIC" or "PEARL" powders
- Guarantees a fast, efficient procedure for greater working comfort



Clinical case Dr P. ROUAS (France)

AIR-N-GO® PERIO EASY NOZZLE *(optional PERIO easy kit)*

- Innovative design in a "shoehorn" shape providing easy sub-gingival insertion
- A real asset for non-surgical, sub-gingival treatment of teeth and implants (3-8mm periodontal pockets)
- 5 seconds per face is all that is needed



Clinical case Dr C. LIM (U.K.)

AIR-N-GO® PERIO MAINTENANCE NOZZLE *(optional)*

- Reduced consumption of AIR-N-GO® "PERIO" fine powder particles in the maintenance phase
- Treatment of mucositis and return to a healthy clinical situation
- Action against the progression of periodontal disease



Clinical case Dr C. LIM (U.K.)

AIR-N-GO® PERIO NOZZLE *(optional)*

- Sub-gingival use for surgical and non-surgical treatments (tunnelling and open flaps)
- Powerful double lateral spray
- Decontamination of deep periodontal pockets (8-10mm)



Clinical case Dr A. PATEL (U.K.)



CUSTOMISED SOLUTIONS FOR YOUR CLINICAL NEEDS

AIR-N-GO® powders have been specially designed and studied to produce effective treatment that is gentle on all treated surfaces. All these powders are compatible with the AIR-N-GO® easy, no change of handpiece is required.

AIR-N-GO® "CLASSIC" POWDERS


Sodium bicarbonate-based
($\approx 76\mu\text{m}$)

- **Complete range of active supra-gingival powders** which meet any patients' preferences
- Scrupulous cleaning giving instant results
- Action of removing coloured stains and bacterial plaque, even in areas that are difficult to access
- Better patient comfort: gentle and efficient with limited bleeding during treatment




5 fresh flavours based on 100% natural aromas or essential oils:
Neutral, Cola, Raspberry, Peppermint and Lemon

COMPETITOR

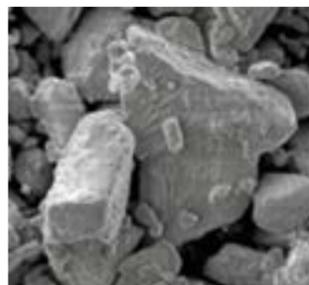


Stratified particles with aggressive geometry (x1000)

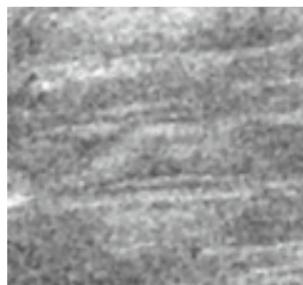


Treated enamel shows numerous scratches (x300)

AIRN GO

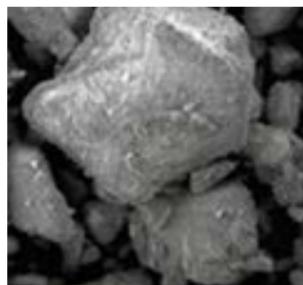

Particles with softened geometry (x1000)

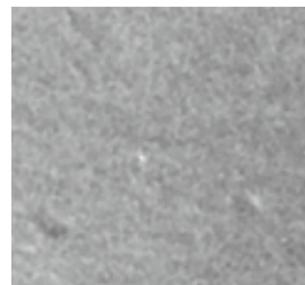

Treated enamel: surface preserved without impacts or scratches (x300)

Source: ICMCB CNRS (Institute of Condensed Matter Chemistry - French National Centre for Scientific Research) S.E.M. photos


AIR-N-GO® "PEARL" POWDER

Natural calcium carbonate-based
($\approx 55\mu\text{m}$)



- Supra-gingival polishing powder **for delicate tissues protection**
- Reduction in pH acidity caused by tissue inflammation; return to clinically healthy buccal flora
- Restoration of the natural shine and pearliness of dental enamel
- Fast cleaning of difficult stains and composites in total safety

Even the most sensitive patients with advanced gingivitis will benefit from "PEARL" microspheres.

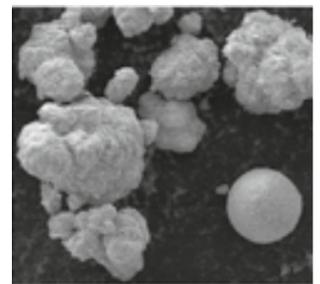

AIR-N-GO® "PEARL" (x1000)

Source: ICMCB CNRS (Institute of Condensed Matter Chemistry - French National Centre for Scientific Research) S.E.M. photos

AIR-N-GO® "PERIO" POWDER

Natural glycine-based
($\approx 25\mu\text{m}$)



- **Exclusive use for sub-gingival treatments** as a complement to the initial therapy
- Low abrasiveness preserving the treated surfaces (teeth and implants) and the most fragile anatomical areas
- Prevents the progression of the sub-gingival biofilm

Multiple advantages with unique properties:

- Plaque removal: during periodontal maintenance on exposed root surfaces without sacrificing substance
- Reduction of the bacterial load in the periodontal pockets



AIR-N-GO® "PERIO" (x1000)

Source: ICMCB CNRS (Institute of Condensed Matter Chemistry - French National Centre for Scientific Research) S.E.M. photos