MI Varnish IFU WEP
30000709_NZ6704

Prior to use, carefully read the
instructions for use.

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ADVERSE REACTIONS

Eodematus swelling has been reported only in rare instances

in some fluoride varnish products, especially after application to extensive surfaces. Although extremely rare, dyspnea has occurred in asthmatic children. Nausea has been reported when extensive applications have been made to patients with sensitive digestive systems.

If required, varnish film can be removed by thoroughly brushing once set.

FLAVOR

Fresh Strawberry Fresh Mint

STORAGE

Store in a cool, dry and low humidity area away from direct sunlight (8-25°C).

PACKAGE

MI Varnish Intro pack, 10 unit doses MI Varnish (5 Strawberry + 5 Mint)
MI Varnish Refill pack, 35 unit doses MI Varnish Strawberry
MI Varnish Refill pack, 35 unit doses MI Varnish Mint
MI Varnish Clinic pack, 100 unit doses MI Varnish Strawberry
MI Varnish Clinic pack, 100 unit doses MI Varnish Mint
One dose package contains

Note:

1 mL of MI Varnish contains 50 mg of sodium fluoride (22.6 mg fluoride ion).

CAUTION

1. MI Varnish is flammable. Do not use near open flame. Keep away from sources of ignition. Do not store large quantities in one area. Keep away from direct sunlight.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. Avoid swallowing or digestion of material.

4. MI Varnish unit dose and brush are disposable. Do not reuse as this might cause cross contamination between patients.

5. In case of contact with clothes, table or floor, immediately wipe off with ethanol.

6. Do not mix with other products.

7. Dispose of all waste according to local regulations.

8. MI Paste PlusTM should not be used on the same day but can be used from the following day. GC Tooth Mousse (without Fluoride) can be used four hours after application of MI Varnish.

Some products referenced in the present IFU may be classified as hazardous according GHS. Always familiarize yourself with the safety data sheets available at <http://www.gc-europe.com>. They can also be obtained from your supplier.

NOTE

Recaldent (CPP-ACP) technology was developed by School of Dental Science, The University of Melbourne, Australia.

*RECALDENT is a trademark used under licence. Recaldent (CPP-ACP) technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

For the Summary of Safety and Clinical Performance (SSCP) please see EUAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gce@gc.dental

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

as well as to our internal vigilance system:

vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

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Recaldent (CPP-0A3C/2P0) technology was developed by School of Dental Science, The University of Melbourne, Australia. *RECALDENT is a trademark used under licence. Recaldent (CPP-ACP) technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.