


Variolink® Esthetic DC


Placement of inlays (glass ceramic, composite)

Try-in


Variolink
Esthetic
Try-in

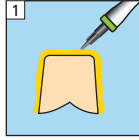


Application of
Try-in paste

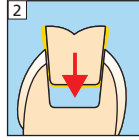


Try-in of restoration

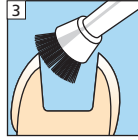


Cleaning after Try-in

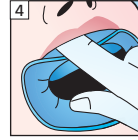


Isolation

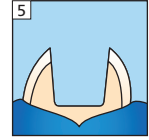

Restoration

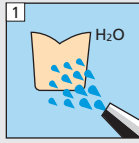

H₂O

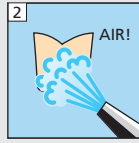

AIR!

Pre-treatment of ceramic restorations:

According to manufacturer's instructions

E.g. IPS Empress: Etch for 60 s


IPS e.max CAD/Press: Etch for 20 s
with IPS Ceramic etching gel (hydrofluoric
acid). Follow safety instructions.

Pre-treatment of composite restorations:

According to manufacturer's instructions

E.g. SR Nexco, 2 bar, Al₂O₃ 100 µm
or roughened with finishing diamonds.

Monobond
Plus

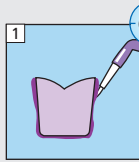


60 s




AIR!

Total Etch
optional


Preparation

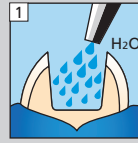

H₂O

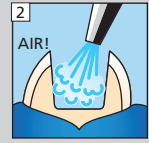


AIR!

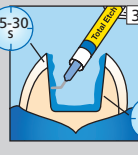


15-30 s

3

15 s

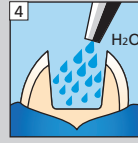


H₂O

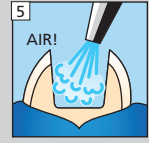

AIR!



Adhese
Universal

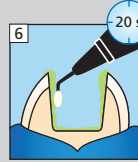

20 s




AIR!

Avoid pooling





Bluephase
Style

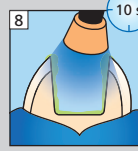


10 s

Cementation

Variolink
Esthetic DC

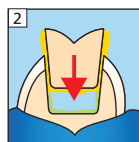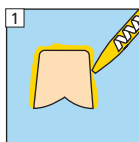


Bluephase
Style

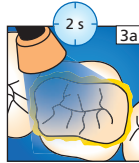


2 s

3a

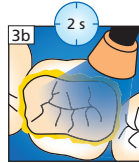


2 s

3b

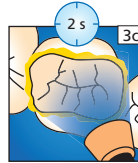


2 s

3c

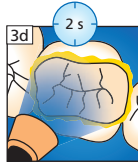



2 s

3d

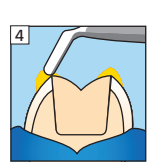


Secure the restoration until final polymerisation

Liquid
Strip

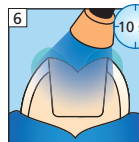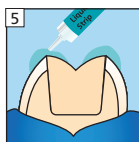


* per segment and
mm ceramic

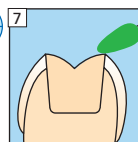

Polishing

Fluor
Protector

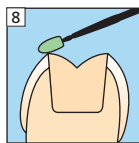

more information
www.cementation-navigation.com

CE 0123



Please read
Instruction for Use

ivoclar
vivadent
clinical

Date information prepared: 2014-06-25
666357/e/Rev. 0

Variolink® Esthetic DC


Placement of crowns and bridges
(glass ceramic, oxide ceramic, composite)

Try-in

Variolink
Esthetic
Try-in



Restoration

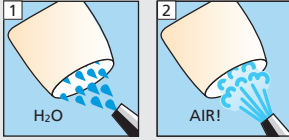


Pre-treatment of glass ceramic restorations:
According to manufacturer's instructions
E.g. IPS Empress: Etch for 60 s
IPS e.max CAD/Press: Etch for 20 s
with IPS Ceramic etching gel (hydrofluoric acid). Follow safety instructions.

Pre-treatment of oxide ceramic restorations:
According to manufacturer's instructions
E.g. IPS e.max ZirCAD
max. 1 bar, Al₂O₃ 100 µm




Pre-treatment of composite restorations:
According to manufacturer's instructions
E.g. SR Nexco, 2 bar, Al₂O₃ 100 µm
or roughened with finishing diamonds.

Monobond
Plus

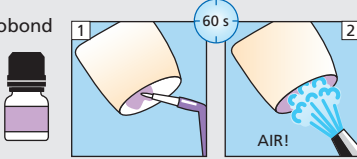


Total Etch
optional


Preparation

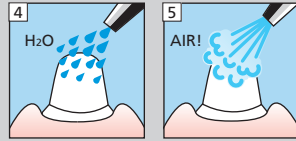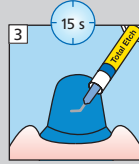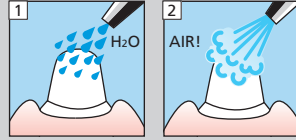

Adhese
Universal

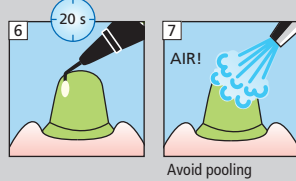


Bluephase
Style

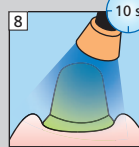


Cementation

Variolink
Esthetic DC

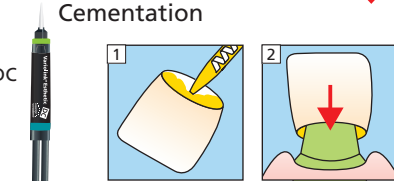


Bluephase
Style

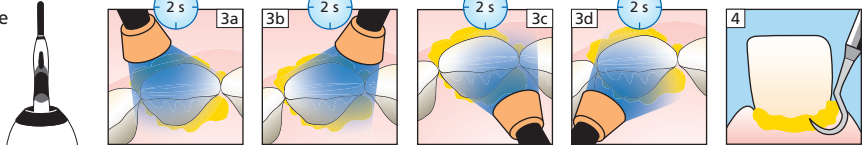


Secure the restoration until final polymerisation

Liquid
Strip

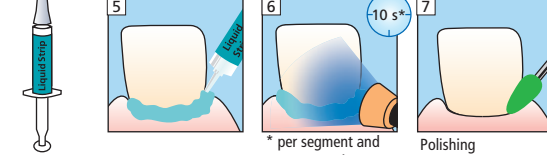

* per segment and
mm ceramic

Fluor
Protector

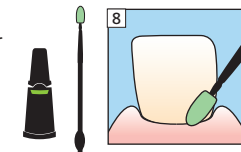

more information
www.cementation-navigation.com

CE 0123

Please read
Instruction for Use

ivoclar
vivadent
clinical