



scaling



Supra-gingival scaling



1

Universal tip

Simple cases: gross supra-gingival scaling.

*Tangential orientation to the surface.
To-and-fro sweeping to "detach" the tartar whilst respecting the enamel.*



2

Voluminous calculus

Removal of significant supra-gingival deposits.

Apply the flat part to the tooth surfaces.



3

Stains

Removal of marks and stains (tobacco, tea, coffee, etc.).

Apply the rounded extremity of the tip to the surface to be treated.



Sub-gingival scaling and probing



10P

Shallow pockets

Scaling of pockets less than 2-3mm deep.



10Z

Medium pockets

Scaling of medium pockets (< 4mm).

Removal of biofilm and soft deposits, while evaluating the depth of the pockets using the marks every 3mm.

Efficient for maintenance treatment in patients with good dental hygiene.