

hydrocolor^{COOL}**5**



CHROMATISM & STABILITY

Clinical

Chromatic High Stability Alginate

Zhermack[®]

CLINICAL



hydrocolor 5

The chromatic alginate that remains stable for 5 days

ZHERMACK have created a new *chromatic alginate* with **5 days dimensional stability** of the impression to simplify dentists' everyday work.

3 COLOURS IN SYNCHRONY

HYDROCOLOR 5 changes colour to indicate to the dentist the time required for impression taking. Fuchsia indicates the **mixing time**, a change into violet means **working phase**, during which the impression tray can be loaded and positioned in the patient's mouth, where it will then turn light blue and remain this colour until **fully set**.



Mixing phase



Working phase



Positioning in the mouth

TECHNICAL DATA

Mixing time	<30sec
Working time*	1min 10sec
Time in mouth	1min
Setting time	2min 10sec

Elastic recovery (ISO 1563)	97%
Permanent deformation (ISO 1563)	12%
Compressive strength (ISO 1563)	1.1 MPa

* Includes mixing time

COOL SENSATION

HYDROCOLOR 5 is *consistent and creamy* at the same time, for maximum patient *comfort*.

Its exclusive *cool berry flavour* makes **HYDROCOLOR 5** particularly indicated for younger patients, as it reduces the stress involved during impression taking and leaves a pleasant *fresh sensation* in the patient's mouth.



MAXIMUM STABILITY, ABSOLUTE SAFETY

Clinical practice often involves too short a session to cast the model immediately; impressions taken with **HYDROCOLOR 5** maintain their *dimensional stability* for **5 days****, offering more *flexibility* so you can organise your work without jeopardising the results.

Prior to storing the impression or casting the model, **ZHERMACK** recommends *disinfecting* it with **ZETA 7 SPRAY** or **ZETA 7 SOLUTION**, products especially created to protect against cross contamination *without altering the quality of the impression*.



** When stored in a hermetically sealed bag at 23°C/73°F

hydrocolor^{COOL}5



Packaging

C302120 HYDROCOLOR 5	1 bag 453 g
C302125 HYDROCOLOR 5 INTRO KIT	2x453 g bags + measures + container
C302130 HYDROCOLOR 5	1 single dose sachet, 21 g
C302070 HYDROGUM 5	1 bag 453 g
C302071 HYDROGUM 5 INTRO KIT	2x453 g bags + measures + container
C302075 HYDROGUM 5	1 single dose sachet, 21 g
C302025 HYDROGUM	1 bag 500 g
C302041 HYDROGUM INTRO KIT	2x500 g bags + measures + container
C302051 HYDROGUM	1 single dose sachet, 18 g
C302145 ORTHOPRINT	1 bag 500 g
C302161 ORTHOPRINT INTRO KIT	2x500 g bags + measures + container
C302171 ORTHOPRINT	1 single dose sachet, 18 g
C302240 TROPICALGIN	1 bag 453 g
C302242 TROPICALGIN INTRO KIT	2x453 g bags + measures + container
C302245 TROPICALGIN	1 single dose sachet, 18 g
C302086 PHASE PLUS	1 bag 453 g
C302111 PHASE PLUS	1 single dose sachet, 18 g
C302205 NEOCOLLOID	1 bag 500 g
C302221 NEOCOLLOID INTRO KIT	2x500 g bags + measures + container
C302230 NEOCOLLOID	1 single dose sachet, 18 g

Accessories

C300990	Plastic alginate spatulas set (x6)
C300992	Flexible rubber mixing bowl
C300960	Long life bags (x100)
C400431	ALGITRAY LIQUID detergent for removing alginate from the impression tray. 2 l bottle
C400430	ALGITRAY POWDER 1 kg tub + measure
C300900	1 set of measures for Hydrocolor 5 and Hydrogum 5 (1 water + 1 powder)
C300910	1 set of measures for other alginates (1 water + 1 powder)

Equipment

C303230 ALGHAMIX II	Automatic alginate mixer (230 V)
C303215 ALGHAMIX II	Automatic alginate mixer (115 V)

Equipment accessories

XR0250600	350 cc Alginate bowl - light blue
XR0250601	350 cc Alginate bowl - orange
XR0250602	350 cc Alginate bowl - navy blue
XR0250603	350 cc Alginate bowl - green
XR0250604	350 cc Alginate bowl - white
XR0250606	350 cc Alginate bowl - grey
XR0250650	600 cc Gypsum bowl - grey
C303030	20 sheet mixing pad for Alghamix II
XR0250670	Mixing disk for Alghamix II + 3 pads
C207200	Silicone mixing spatula

BRANCH

GERMANY
Zhermack GmbH
Deutschland
Öhlmühle 10 - D-49448 Marl
Tel. +49 - 0 54 43 / 20 33 - 0
Fax +49 - 0 54 43 / 20 33 - 11
info@zhermack.de
www.zhermack.com

POLAND
Zhermapol Sp. z o.o.
ul. Augustówka 14
02 - 981 Warszawa - Polska
Tel. +48 - 22 858 82 72
+48 - 858 73 41
Fax +48 - 22 642 07 14
zhermapol@zhermapol.pl
www.zhermack.com

USA
Zhermack Inc.
P.O. Box 4195, River Edge
New Jersey 07661 - 4195
Tel. +1 (877) 819 6206
+1 (732) 389 8540
Fax +1 (732) 389 8543
info@zhermackusa.com
www.zhermackusa.com

COUNTRY CONTACTS

UK AND IRELAND
Tel. +44 - (0)7870 690811
uk@zhermack.com

FRANCE
Tel. +0800 - 915083
info.france@zhermack.com

MEXICO AND CENTRAL AMERICA
Tel. +52 - 1 - 55 - 91980497
info.mexico@zhermack.com

SPAIN
Tel. 900 99 39 52
info.spain@zhermack.com

RUSSIA
Tel. +7 916 930 1191
info.russia@zhermack.com

SOUTH AMERICA
Tel. +57 - 310 - 2601 777
info.colombia@zhermack.com

Zhermack SpA

Via Bovazecchino, 100 - 45021 Badia Polesine (RO) ITALY
Tel. +39 - 0425 597611 Fax +39 - 0425 53596 info@zhermack.com www.zhermack.com

Zhermack[®]
BEYOND INNOVATION