



# periodontics



## Periodontal debridement



**H3**  
F00369



### Initial periodontics, anterior sector

Treatment of the incisor-canine block.

*The guide edge is oriented parallel to the pocket. The H3 tip is descended into the periodontal pocket without risk of injury to the ligament. The cavitation will lift the debris out.*



### Periodontics for the premolar and molar sectors, left-oriented

First instrument in the sequence for treating all the surfaces and the furcations.

- Maxillary: buccal and distal surfaces of sector 2, pivots at 13, then the buccal and mesial surfaces of sector 1.
- Mandibular: buccal and distal surfaces of sector 4, pivots at 43, then lingual and mesial surfaces of sector 3.



### Periodontics for the premolar and molar sectors, right-oriented

Second instrument in the sequence.

- Maxillary: palatine and mesial surfaces of sector 2, pivots at 13, then buccal and distal surfaces of sector 1
- Mandibular: lingual and mesial surfaces of sector 4, pivots at 43, then buccal and distal surfaces of sector 3.



**H4L**  
F00114



**H4R**  
F00115