

Cleaning and Sterilization Instructions for medical devices

Cleaning:

1. Remove visible debris or organic matter with a disposable wipe or surface brush. This has to be done by scrubbing with warm tap water and soap or detergent.
2. Inspect instruments for debris on surfaces. Repeat cleaning step 1 if there are still debris.
3. Immediately soak in an appropriate disinfecting solution, following the manufacturer's instructions.
4. Alternatively to point 3, an ultrasonic bath cleaner with an appropriate cleaning and disinfecting agent can be used, in accordance with the manufacturer's instructions. Please assure that the medical device has the relevant symbol on the packaging.
5. Rinse the medical device with warm tap water.
6. Alternatively to points 1 to 5, a thermo disinfection with an appropriate cleaning agent can be used in accordance with the manufacturer's instructions. Please assure that the medical device has the relevant symbol on the packaging.
7. Dry the medical device and place it in an autoclave tray, or in a sterilization pouch (see Sterilization instructions step 1 below).



Sterilization:

1. Place the medical device in an autoclave tray, or in a sterilization pouch.
2. Sterilize in a steam autoclave using distilled water and at the temperature specified on the relevant symbol.
3. Do not use chemical autoclave or cold sterilization solutions.
4. Avoid direct contact with the metallic part of the autoclave.

DISCLAIMER:

This product is of high quality, manufactured by Polydentia SA, in Mezzovico, Switzerland.

Although the materials used in the manufacturing process are designed with durability and performance in mind, please note that all plastic materials have a limited lifespan.

The product in this packaging has been inspected, tested and approved by our quality officers. All plastic materials alter, discolor, and deteriorate over time, and can break when repeatedly exposed to chemical disinfectants, detergents, and repeatedly sterilized. Polydentia SA will not be able to address any requests for refund or exchange of any products which have not been reprocessed and sterilized in accordance with these instructions for use.