


Professional-quality products for at-home use

Ivoclar Vivadent, headquartered in Liechtenstein, has been developing and delivering innovative solutions for the protection and restoration of oral health since 1923. In more than 120 countries around the world, dentists and dental technicians are treating their patients with the clinically proven, high-quality products from Ivoclar Vivadent.

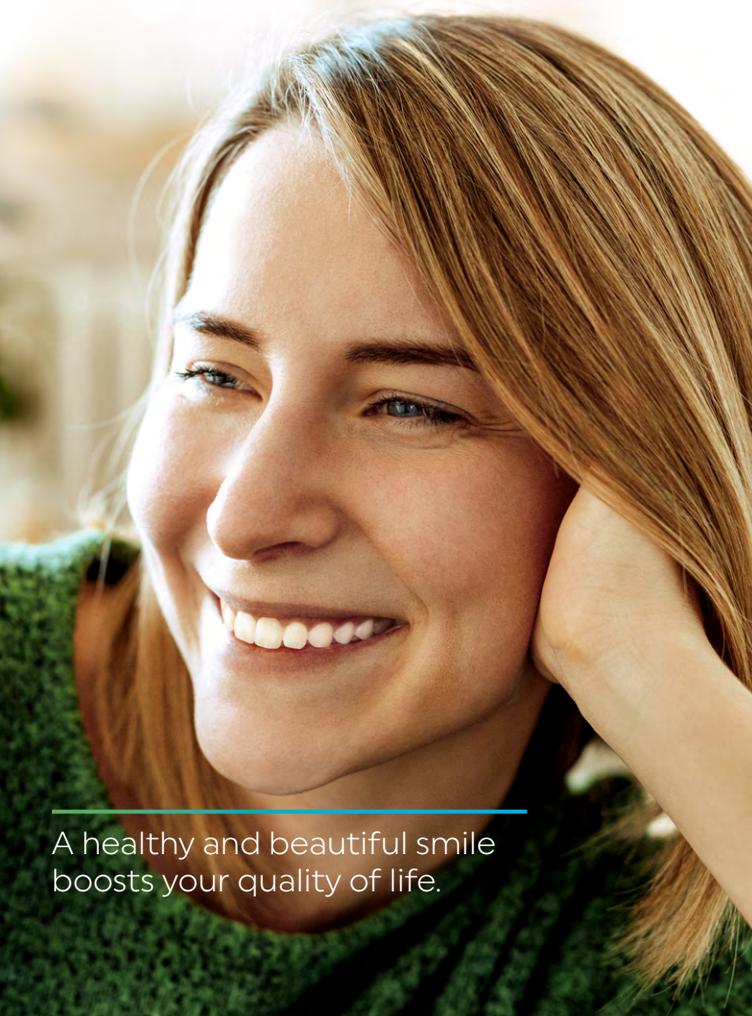

A healthy and beautiful smile boosts your quality of life.



Professional

Specially recommended for you by your dental practitioner

Effective

Clinically proven product and visible results

Easy

Application with the supplied brush

Flexible

Selective application on individual teeth

Cost-effective

Without any specially fabricated tray

Ivoclar Vivadent AG
Bendererstr. 2
9494 Schaan
Liechtenstein
Tel. +423 235 35 35
Fax +423 235 33 60

ivoclarvivadent.com

**ivoclar
vivadent**

Paint on a bright new smile!



**VivaStyle[®]
Paint On Plus**

No tray. No strips. Professional teeth whitening at home.

Making People Smile

**ivoclar
vivadent**

Five steps at home for a new smile

VivaStyle Paint On Plus is a teeth whitening varnish system. It is applied on the teeth with a brush. The proven ingredient hydrogen peroxide releases oxygen, which gently lightens stains.

1

Clean
teeth thoroughly

2

Dry
with a paper towel

3

Apply
with the supplied brush

4

Wait
30 sec to dry
10 min to take effect

5

Remove
varnish film with a toothbrush

Finished!

Used for 2 weeks, once daily – results will be visible after only 7 days

Frequently asked questions about teeth whitening

What causes stains on teeth?

Stains and discolouration are classified into two types: external and internal. External stains may be caused by the consumption of dark beverages such as coffee, tea and red wine as well as by smoking or taking certain medications. Pigments accumulate on the tooth surface, which stain the tooth structure and form plaque if oral hygiene practices are inadequate.

How long will the whitening results last?

Depending on your lifestyle, the whitening results will last up to six months at the least. Your diet (consumption of e.g. coffee, red wine or tea), your age and your dental hygiene regime as well as smoking habits decisively influence the duration of the results. Nevertheless, the treatment can be repeated if necessary.

Should I expect any side effects?

If varnish should come in contact with oral tissue, you may feel a brief cold sensation. In some cases, you may experience temporary sensitivity of teeth or irritation of oral tissue when you use the varnish. If irritation persists, please contact your dentist.

What needs to be considered before using the product at home?

It is absolutely essential for your dentist to examine you first and discuss a treatment plan with you. Then you can carry out the treatment at home according to the instructions of your professional dental team up until you are due for the next examination.

Can anyone use the varnish?

Teeth whitening is suitable for people who are older than 18 years of age. The varnish is not suitable for women who are pregnant or breastfeeding.