Enamelast® Fluoride Varnish

5% Sodium Fluoride Varnish

Description: Enamelast® Fluoride Varnish is a flavored, xylitol-sweetened 5% Sodium fluoride in a resin carrier delivered in a 1.2 mL syringe or in a 0.4 mL unit dose blister pack.

Indications for Use:

Enamelast® Fluoride Varnish is 5% sodium fluoride in a varnish carrier which produces a mechanical occlusion of the dental tubules in the treatment of tooth hypersensitivity.

Directions for Use:

- Syringe Delivery
1. Thoroughly brush teeth, rubber cup polish or wipe teeth with gauze prior to placement.
- Remove the Luer Lock Cap from the syringe and securely attach the SoftEZ™ delivery tip (sold separately).
NOTE: Enamelast can also be placed on the back of the clinician's gloved hand or in a dappen dish and applied with a standard brush (sold separately).
- Verify flow prior to applying intraorally.
- Verify the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
- Lightly dry area to be treated.
- Lightly dry area to be treated.
- Using a painting motion, apply a thin smooth layer to as many dry tooth surfaces as possible.

Unit Dose Delivery

- Thoroughly brush teeth, rubber cup polish or wipe teeth with gauze prior to placement.
- Open blister pack by peeling back and removing foil lid.
- Using the disposable delivery brush, lightly stir the varnish.
- Lightly dry area to be treated.
- Using a painting motion, apply a thin, smooth layer of Enamelast® to as many dry tooth surfaces as possible.

NOTE:

- The brush may be bent up to a 90- degree angle for easier access.

NOTE:

- Enamelast sets when it contacts with water or saliva. Allow cheeks, lips, and saliva to contact teeth or gently flow cool water over the teeth. For esthetic reasons, encourage patients not to "lick" their teeth for a few minutes after placement.
- Patients should avoid brushing, flossing and chewing hard, sticky and hot foods for 4–6 hours after treatment.

Enamelast® Fluoride Varnish will begin to flake off with each brushing.

Precautions and Warnings:

- Carefully read and understand all instructions before using.
- Do not use more than half of the 1.2 mL syringe of fluoride varnish per patient per application.
- Store at room temperature.
- Verify syringe flow prior to applying intraorally. If resistance is met, replace tip and re-check. Use only recommended tips.
- Do not remove cap from the syringe or foil lid from the unit dose until ready to use.
- Patient, clinician, and assistant must wear protective eyewear when using this product.
- Do not use on patients with ulcerative gingivitis or stomatitis.
- Do not use with other topical fluoride treatments the same day and instruct patient to refrain from using oral fluoride supplements for 4 (four) days after treatment.
- Do not use on patients with a known sensitivity to ingredients including colophony (kolophonium) and/rosin. If allergic reaction, dermatitis, or rash develops, consult a physician.
- In rare instances, edematous swelling has been reported after extensive fluoride varnish treatment and dyspnea has been reported in asthmatic children.
- Some patients may experience nausea after application. If nausea persists, thoroughly brush to remove the varnish film.
- Syringe tips are for single- use only, to avoid cross contamination. The prefilled syringes can be used repeatedly. Use Syringe Covers to avoid cross contamination. After use, remove and dispose the Syringe Cover, together with the syringe tip. Re-cap the syringe with the Luer Lock Cap and disinfect the syringe with a disinfecting agent, suitable for medical devices, which has no protein-filing effect.
- Dispose of containers, brushes, tips and empty syringes properly.
- Keep out of reach of children.
- For professional use only.

NOTE: For SDS and additional information about using our products, please go to www.ultradent.com

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2. 不得在口腔内吮吸或者类似使用。	2. Do not suck or use in a similar way in the mouth.
3. 不得与食物或其他外用药物同时使用，嘱咐病人用药4(四)天内不得要补充口服液体。	3. Do not use with food or other external drugs at the same time. Advise the patient not to take oral fluids for 4 (four) days after using.
9. 已知对过敏性体质患者，包括对豚鼠 (kolphonium) 和/或松香过敏，严禁使用本产品。如发生过敏反应，发现皮疹或皮疹，请咨询医师。	9. For patients known to be allergic, including to guinea pig (kolphonium) and/or rosin, the use of this product is strictly prohibited. If an allergic reaction occurs, such as skin rash or skin rash, please consult a doctor.
10. 在极少数情况下，可能出现过敏反应并出现水肿，也有哮喘/儿童哮喘发作的危险。	10. In a few cases, allergic reactions may occur and edema may occur, and there is also a risk of asthma/asthma attack in children.
11. 有的患者治疗后会感到窒息，如果持续窒息，应彻底清除除喉膜。	11. Some patients may feel suffocated after treatment. If the suffocation persists, the pharyngeal membrane should be completely removed.
12. 为避免交叉污染，注射输送头可能配备注射器使用。	12. To avoid cross-contamination, the injection delivery head may be used with a syringe.
13. 本产品为一次性使用，使用后，请丢弃到指定的医疗废弃物收集及交换箱。使用后，移除并弃用注射防护套和注射器。请重新消毒注射器的Luer锁扣并弃用注射防护套和注射器。医用药用新蛋白固定化处理。	13. This product is for single use. After use, please discard it in the designated medical waste collection and exchange box. After use, remove and discard the injection protection sleeve and syringe. Please re-disinfect the Luer lock of the syringe and discard the injection protection sleeve and syringe. The medical use of the new egg white fixation treatment.
14. 正确处理用过的容器、刷子、输送头和针筒。	14. Properly dispose of used containers, brushes, delivery heads and syringes.
15. 避免儿童接触。	15. Avoid children's contact.
16. 仅供专业人士操作。	16. For professional use only.
注意：使用本产品时的SDS和其他信息，请登录www.ohsaeq.com查询。	Attention: For SDS and other information when using this product, please log on to www.ohsaeq.com for query.

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www.ultradent.com

For product SDS please see our website: www.ultradent.com
1-800-552-5512; 801-572-4200

For immediate reorder and/or complete descriptions of all Ultradent's product line, refer to Ultradent's catalog or call toll free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit www.ultradent.com

Enamel+ Fluoride Varnish
5% Sodium Fluoride Varnish

Manufactured by:

ULTRADENT
Ultradent Products, Inc.
5505 West Ultradent Drive (10200 South)
South Jordan, Utah 84095 USA
Made in USA

ULTRADENT
Ultradent Products GmbH
Am Westhofen 30
51149 Cologne Germany

ULTRADENT
ULTRADENT
PRODUCTS, INC.

Copyright © 2017 Ultradent Products, Inc. All Rights Reserved. 10403382 051817