VOCO

Provicol® QM Plus

CE 0482

EN Instructions for use
MD EU Medical device
In accordance with DIN EN ISO 3107

Product description:

Provicol QM Plus is an easy-to-use eugenol-free temporary luting cement, which does not affect the setting of composite-based restoratives and luting materials. The calcium hydroxide contained in **Provicol QM Plus** helps maintain tooth vitality.

Working time: ≥ 1 min (23 °C)
Setting time: ≤ 4 min (37 °C)

Indications:

- Temporary luting of provisional restorations (crowns, bridges, inlays and onlays)
- Temporary luting of definitive restorations (crowns, bridges, inlays and onlays)
- Temporary obturation of small single-surface cavities

Contraindications:

Provicol QM Plus contains modified colophony. **Provicol QM Plus** should therefore not be used in patients with a known hypersensitivity (allergy) to this ingredient.

Patient target group:

Provicol QM Plus is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Provicol QM Plus should only be used by a professionally trained dental practitioner.

Use:

Usable mixing tips: VOCO mixing tip type 10.

1. Preparation:

Ensure that the surfaces to be cemented are clean. Bring material to ambient temperature before applying!
Remove the protective cap from the QuickMix syringe. Due to technical reasons, there may be slight differences in the fill levels of both chambers prior to initial use. You should therefore remove the cap and continue to press out the material until an equal amount has been squeezed out of both outlets. Then attach a mixing tip and lock (with a 90° turn clockwise). The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories. The attached mixing tip is intended for single use only. After use, store the QuickMix syringe tightly sealed. Check that material flows freely through the openings before reusing.

2. Placing the restoration/application:

Apply **Provicol QM Plus** to the dry temporary restoration and/or the prepared tooth surface. Then place the restoration by applying gentle pressure, generating excess material in the process. The working time at room temperature is at least 1 min.

3. Curing:

Any excess material can be removed using the following methods:

- Remove excess amounts of not fully cured **Provicol QM Plus** using a foam pellet or similar.
 - Wait for material to cure fully (observation of progress in the mouth) and then remove excess cured cement using a suitable instrument.
- Contact points and interproximal areas need to be cleaned with dental floss. The sulcus must be free of material residues, as they may irritate the gingiva. Curing is completed 4 minutes after insertion. The restoration must be held in position during the polymerisation phase.

4. Removal:

Use a suitable instrument to remove the restoration and completely remove all **Provicol QM Plus** residues before final luting. This can be done with a suitable instrument (curette, scaler, probe) and, if required, with polishing paste.

Warnings, precautionary measures:

- In case of contact with the eyes, immediately rinse with plenty of water and seek medical advice.
- The sensitivity of cores can be reduced by using desensitising preparations prior to temporary luting.
- Appropriate measures for pulp protection are recommended in cases of exposed pulp or insufficient residual dentine thickness.
- A harder cement is the better choice for multi-surface cavities and those which have an extensive surface area, as well as long-term temporary obturations.
- **Provicol QM Plus** residues are easily cleaned off restorations after removal if they are briefly treated with cooling spray outside the mouth.
- Wetting prepared teeth with a little water before cementation makes subsequent removal of the restoration easier.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:

Store at 4°C-23°C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Provicol QM Plus** must be reported to VOCO GmbH and the responsible authority.

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