

CleanJoy®

FLUORIDE-CONTAINING TOOTH CLEANING AND
POLISHING PASTE IN THREE CLEANING GRADES


Effective cleaning action yet particularly gentle

Basis for the prophylaxis of caries, periodontitis and peri-implantitis, in addition to thorough sound oral hygiene at home, is professional tooth cleaning (PTC) carried out regularly and tailored to individual requirements. CleanJoy was developed to simplify the procedure of tooth cleaning as far as possible and to convey a positive experience to the patient. The prophylaxis and cleaning paste system is available in three pleasant flavours: cherry especially for children, mint as a classic taste and caramel as a creamy version. This system enables you to thoroughly yet gently remove hard as well as soft plaque, and staining, from dental hard substance, fillings, implants and prosthetic restorations. The “fine” paste is barely abrasive and therefore also suitable for polishing after using air polishing systems. The final CleanJoy application thus creates noticeably smooth surfaces which inhibit the renewed colonisation by plaque-forming bacteria and the formation of staining. These are perfect preconditions for long-term oral health.

Handling and consistency

CleanJoy is characterised by its well-adjusted consistency and homogeneity. The prophylaxis paste is stable and at the

same time soft. This means that dosage is exact and the paste does not splash when used, whether working with a small brush or a cup. Homogeneous consistency and stability guarantee good wettability and retention on the surfaces to be treated. Subsequent removal of the paste is also simply done with a water jet.

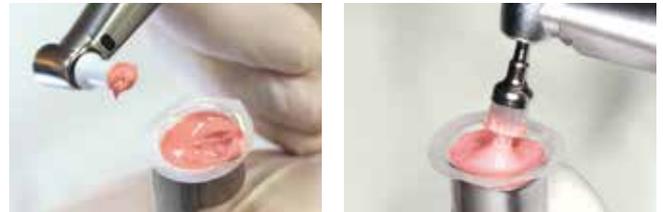

Smooth surfaces, fluoride and xylitol provide the optimal foundation until PTC is next carried out

All abrasion grades of the CleanJoy prophylaxis pastes contain fluoride (700 ppm) and xylitol. The fluoride helps strengthen the dental hard substance and has a remineralising action. Smooth surfaces, in combination with xylitol, inhibit deposition / colonisation of plaque-forming bacteria.



Photo: Bonn Medien

Andrea Beyer, dental assistant specialising in prophylaxis and orthodontics, Münchhausen (Germany): *“Professional tooth cleaning is more and more in demand with patients of all age groups. This is why our team has long been searching for a cleaning paste capable of removing persistent plaque and cleaning the tooth gently. In view of our younger patients we were also hoping to find a prophylaxis paste which can be used without reservations on children in general, as well as particularly on children wearing multiband appliances. A paste which cleans without leaving a sensation of roughness of the surfaces of the teeth or damaging the brackets in any way.*


We have now found CleanJoy by VOCO and its application is fun, with the “traffic light” colour-coding meaning that it can be individually adjusted according to the degree of staining each patient presents. Working from red (for heavier staining) to green (for lesser staining) leaves a velvety sensation in the mouth with no rough areas. Our best experience was using CleanJoy in combination with silicone cleaning cups.”

CleanJoy®

INDIVIDUALLY ADJUSTABLE

CleanJoy – *SingleDose*

The handy *SingleDose* packaging helps extract the paste hygienically and in individual doses. One *SingleDose* contains sufficient material to clean a complete denture, keeping it under seal and fresh until it is used.

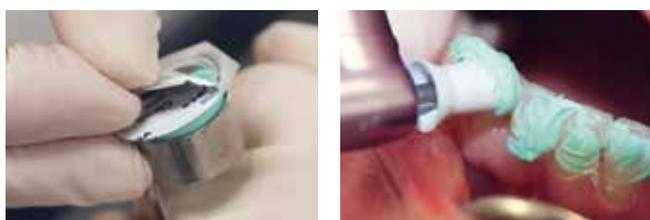

Traffic light coding system – three grain sizes for optimal results

CleanJoy is a cleaning system of three coordinated pastes, which also convince when used individually through their effectivity and gentle action on the dental hard substance. Depending on the severity of the plaque and staining, use

either the “coarse”, “medium” or “fine” paste. In any case, the fine polishing paste needs to be used in a final polishing step. The respective grade of abrasion is coded in a “traffic light system”:

Red paste (coarse – RDA value* = 195): effectively and yet gently removes particularly severe and persistent plaque and staining.

Yellow paste (medium – RDA value* = 127): optimally removes light to medium plaque.

Green paste (fine – RDA value* = 16): this paste removes light plaque. The green paste is required for final polishing following use of the red and/or yellow paste/s. It can further be applied following use of an air polishing device.

* RDA values measured by DIN EN ISO 11609 (DIN for toothpaste)

Case 1



Slight extrinsic staining in the anterior region of the mandible



Use of the yellow cleaning paste



Finishing with the green polishing paste



Clean and smooth tooth surfaces

Case 2



Heavy extrinsic staining in the anterior region of the mandible



Treatment of the tooth surfaces using the red paste



Finishing with the green polishing paste



Clean and smooth tooth surfaces

Source: Dr. Sanzio Marques, Passos, Brazil

CleanJoy®

SIMPLY PRACTICAL

Indications

- Removal of soft and hard surface plaque
- Removal of extrinsic discolouration (e.g. staining from coffee, tea or tobacco)
- Polishing of tooth surfaces as part of professional tooth cleaning, prior to whitening or following the removal of orthodontic devices
- Cleaning and polishing of restorations as part of professional tooth cleaning
- Removal of residues of temporary luting material prior to definitive bonding (cementation)

Advantages

- Stable, homogeneous consistency
- No splashing (2,000 – 3,000 r / min)
- No parabens
- Available in three abrasion grades, thus can be individually adjusted to each situation
- RDA values (measured in accordance with DIN EN ISO 11609)
 - coarse (red) RDA = 195
 - medium (yellow) RDA = 127
 - fine (green) RDA = 16
- Traffic light coding: simple to use without confusion
- Can be used with any conventional instrument (small brush, cup, etc.)
- Contains fluoride (700 ppm) and xylitol
- Three pleasant flavours: mint, caramel, cherry



Presentation

Advantages of the *SingleDose*

- Easy, quick and hygienic
- No additional devices required
- No spilling in any position

Coarse (red)			Medium (yellow)			Fine (green)			
Flavour	Tube 100 g	SingleDose 200 x 2 g	Flavour	Tube 100 g	SingleDose 200 x 2 g	Flavour	Tube 100 g	Tube 4 x 100 g	SingleDose 200 x 2 g
Mint	REF 2092	REF 2098	Mint	REF 2091	REF 2097	Mint	REF 2090	REF 2089	REF 2096
Caramel <small>NEW</small>	REF 6092	REF 6096	Caramel <small>NEW</small>	REF 6091	REF 6095	Caramel <small>NEW</small>	REF 6090	–	REF 6094
Cherry <small>NEW</small>	REF 6100	REF 6104	Cherry <small>NEW</small>	REF 6099	REF 6103	Cherry <small>NEW</small>	REF 6098	–	REF 6102

VOCO GmbH
 Anton-Flettner-Straße 1-3
 27472 Cuxhaven
 Germany
 Freecall: 00 800 44 444 555
 Fax: +49 (0) 4721-719-140
 info@voco.com
 www.voco.dental

Available from:

