



GC Fujirock® EP Classic  
GC Fujirock® EP Premium  
GC Base Stone

GC's family  
of **stones**

Your  
**aesthetics**  
start here

**GC**

Superior quality is what makes the GC family of hard-die stones among the most popular in Europe. Used in the production of all kinds of dental prosthetics, GC stones are specifically designed to produce consistently accurate models and dies, fully meeting the requirements of the modern dental laboratory.

True to the systematic approach of GC - "One proven stone family for all requirements and applications" - we present three stones that can be combined perfectly to create the foundation of your aesthetics:



GC Fujirock® EP **Classic**, a truly all-round stone

GC Fujirock® EP **Premium**, an improved stone for high-performance techniques

GC Base Stone, an exceptionally free-flowing stone for bases with minimal expansion

Each stone comes in a range of colours that allow you to create shade combinations that bring out the best in your work ... and your laboratory.

GC's family of stones ... the foundation for success.

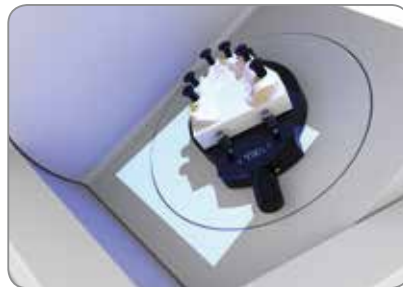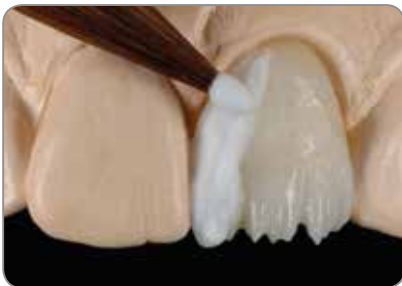



# Three stones, unlin

## GC Fujirock® EP Classic

GC Fujirock® EP Classic lives up to its name. One of the world's most popular Type 4 dental die stones, it is "the classic". It is ideal for all kinds of prosthetics, combining high precision, outstanding edge hardness and high-pressure stability. GC Fujirock® EP Classic is very fluid yet still thixotropic for excellent handling properties and easy pouring with no air bubbles.

Choose the colour that best suits your daily work:

- Golden Brown
- Topaz Beige



## GC Base Stone

GC Base Stone is ideal for basing dentate arches, with either GC Fujirock® EP Classic or Premium, the stones makes them the optimal combination for c

Choose your own complementing colour:

- Terracotta Red
- Royal Blue





# Unlimited possibilities



## GC Fujirock® EP Premium

GC Fujirock® EP Premium has been specially developed for high performance and sensitive technologies such as extended implant cases and complex full-ceramic indications.

The improved powder composition of GC Fujirock® EP Premium and refined colour setting make this special stone the perfect choice for a variety of applications.

Choose the colour that best suits your refined works:

- Pastel Yellow
- Polar White
- Inca Brown
- Titanium Grey



opposing models and study models. When used  
the single "GC quality" standard behind the  
creating superb master models quickly and easily.



# The Art of presenting Aesthetics



# Fujirock® EP Classic & Premium

## Fujirock® EP – Classic

- 890222 Golden Brown 5 Kg pack  
890226 Golden Brown 12 Kg pack  
890225 Topaz Beige 5 Kg pack



| GC Fujirock® EP Classic       |             |
|-------------------------------|-------------|
| Golden Brown / Topaz Beige    |             |
| P/L Ratio                     | 20 ml/100 g |
| Manual mixing                 | 15 sec.     |
| Vacuum mixing                 | 45 sec.     |
| Setting time                  | 10-13 min.  |
| Removal time                  | 30-40 min.  |
| Setting expansion (after 2 h) | ≤ 0.09 %    |
| Compressive strength          | ≈ 53 Mpa    |

## Fujirock® EP – Premium

- 890364 Pastel Yellow 4 Kg pack  
890365 Pastel Yellow 11 Kg pack  
890366 Polar White 4 Kg pack  
890367 Inca Brown 4 Kg pack  
890368 Titanium Grey 4 Kg pack



| GC Fujirock® EP Premium                                  |             |
|--|-------------|
| Pastel Yellow / Polar White / Inca Brown / Titanium Grey |             |
| P/L Ratio  | 20 ml/100 g |
| Manual mixing  | 15 sec.     |
| Vacuum mixing  | 45 sec.     |
| Setting time   | 9-12 min.   |
| Removal time   | 25-35 min.  |
| Setting expansion (after 2 h)                            | ≤ 0.09 %    |
| Compressive strength                                     | ≈ 60 Mpa    |

# Base Stone

## Base Stone

- 890228 Terracotta Red 12 Kg pack  
890229 Royal Blue 12 Kg pack



| Base Stone                    |             |             |
|-------------------------------|-------------|-------------|
| Terracotta Red / Royal Blue   |             |             |
|                               | Flow        | Thixo       |
| P/L Ratio                     | 26 ml/100 g | 21 ml/100 g |
| Manual mixing                 | 15 sec.     | 15 sec.     |
| Vacuum mixing                 | 45 sec.     | 45 sec.     |
| Setting time                  | 9 min.      | 7,5 min.    |
| Removal time                  | 30-40 min.  | 30-40 min.  |
| Setting expansion (after 2 h) | ≤ 0.08 %    | ≤ 0.08 %    |
| Compressive strength          | ≈ 45 Mpa    | ≈ 50 Mpa    |

All packs with easy open / easy close plastic lid. Hermetic seal protects against humidity.

### GC EUROPE N.V.

Head Office  
Researchpark  
Haasrode-Leuven 1240  
Interleuvenlaan 33  
B-3001 Leuven  
Tel. +32.16.74.10.00  
Fax. +32.16.40.48.32  
info@gceurope.com  
http://www.gceurope.com

### GC UNITED KINGDOM Ltd.

Coopers Court  
Newport Pagnell  
UK-Bucks. MK16 8JS  
Tel. +44.1908.218.999  
Fax. +44.1908.218.900  
info@uk.gceurope.com  
http://uk.gceurope.com

